



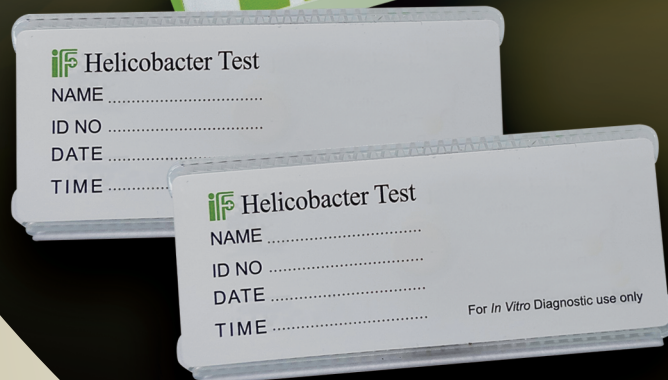


Rapid Helicobacter Disc Test


▶▶▶ Rapid Helicobacter Disc Test

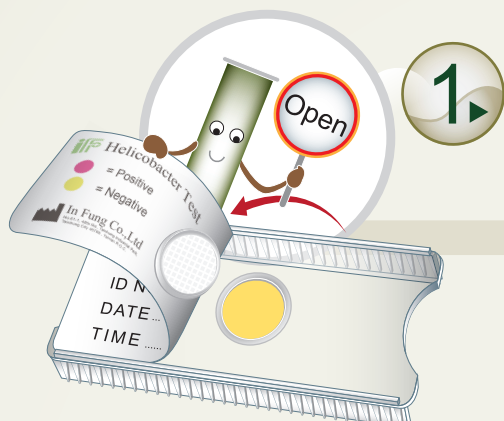


- ▶ Peel the label back to allow access to the disc without removing the whole label.

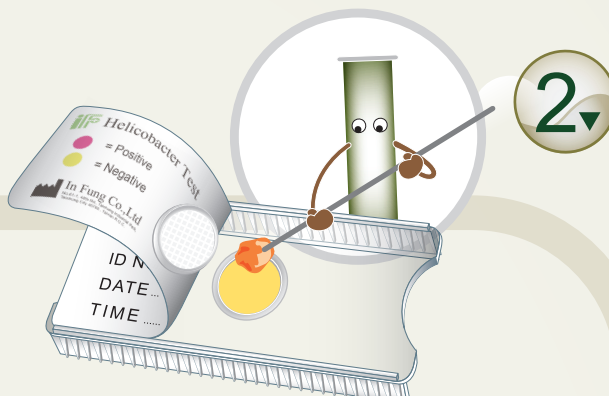


- ▶ Put the biopsy specimen on the disc.

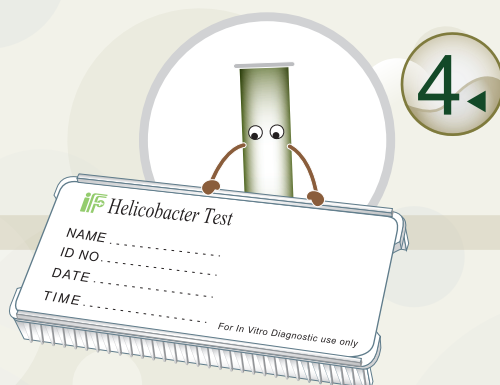

- ▶ Record patient data.

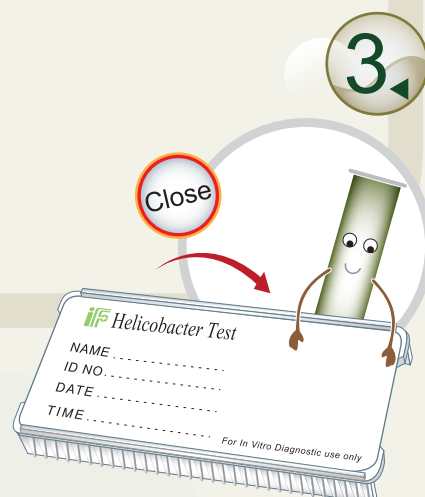


- ▶ Reseal the test slide to cover the disc.

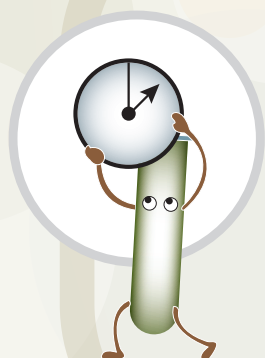

- ▶ Incubate the test slide at room temperature for a maximum of 2 hours.

6

- ▶▶▶ Interpretation of Results after 1 hour of incubation. (In most cases, a positive reaction occurs within 5 to 30 minutes, and all positive results will be completed within 1 hour.)

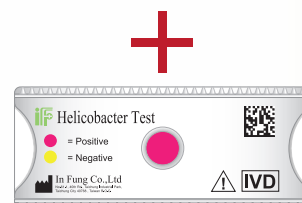

▶ Order Information

Article Number	C0911	C0914	C0926
Description	50 tests/kit	35 tests/kit	70 tests/kit

IN FUNG Co.,Ltd.

- No.61-1, 40th Rd., Taichung Industrial Park, Taichung City 40768, Taiwan R.O.C.
- +886-4-23550886
- service@infung.com.tw
- www.infung.com.tw

