




if
In Fung Co., Ltd

“瑩芳” A/B型流感快速檢測試劑 (未滅菌)
“if” Influenza A&B rapid test (Non-Sterile)
20 tests/kit
許可證字號：衛部醫器製壹字第005789號
產品型號：B0510
保存方式：2-30℃
廠商名稱：瑩芳有限公司
廠商地址：台中市西屯區
製造商名稱：瑩芳有限公司
製造商地址：台中市西屯區

if



Influenza A&B
Extraction Reagent
8 ml.
Use at any temperature from 2°C to 30°C

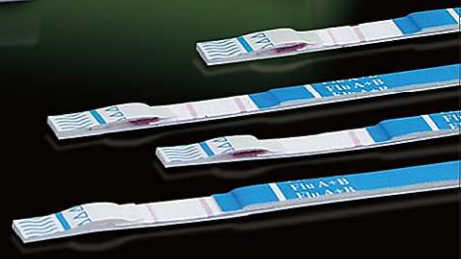

Influenza A&B Rapid Test

瑩芳 A/B 型 流感 快速 檢 試 劑

▶▶▶ Influenza A&B Rapid Test

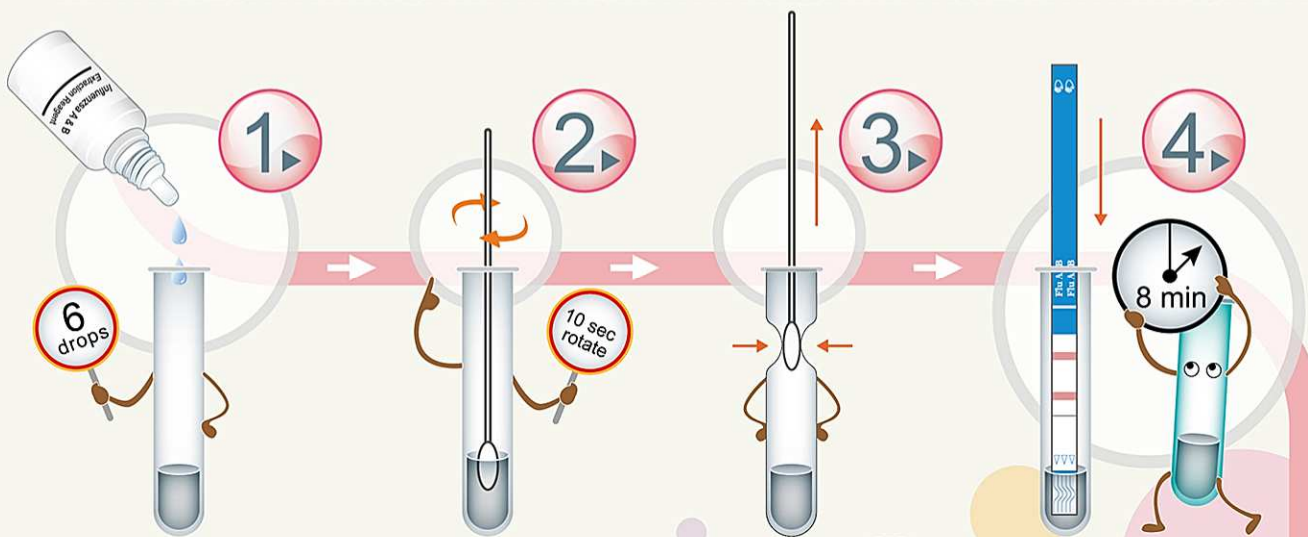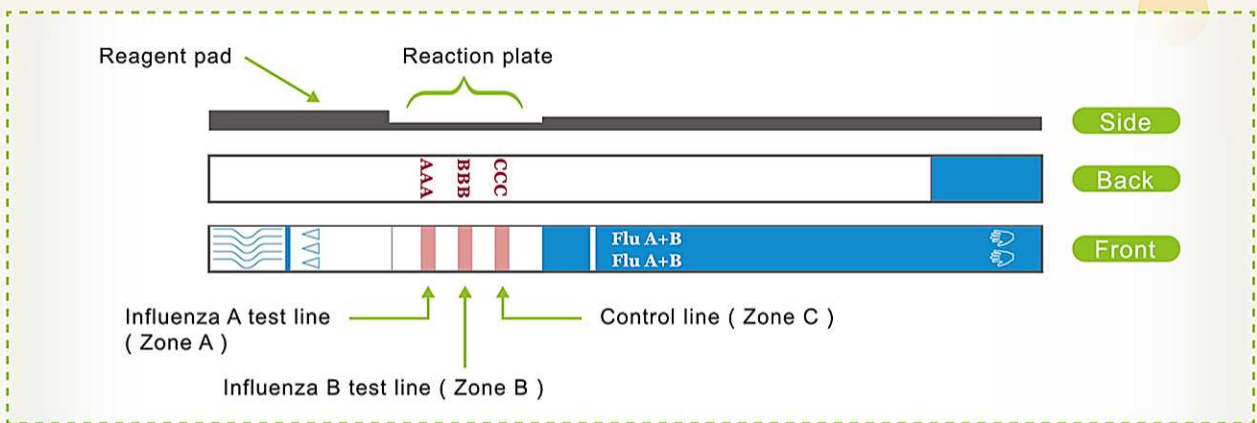


▶ Dispense 6 drops reagent into a test tube without touching the tube edges.

▶ Rotate the swab in the tube for 10 seconds to mix the specimen with the reagent.

▶ Squeeze the specimen with your fingers from the outside of the test tube before pulling it out.

▶ Put the test strip into the tube for 8 minutes and read the results not later than 20 minutes.

▶▶▶ Interpretation of Results

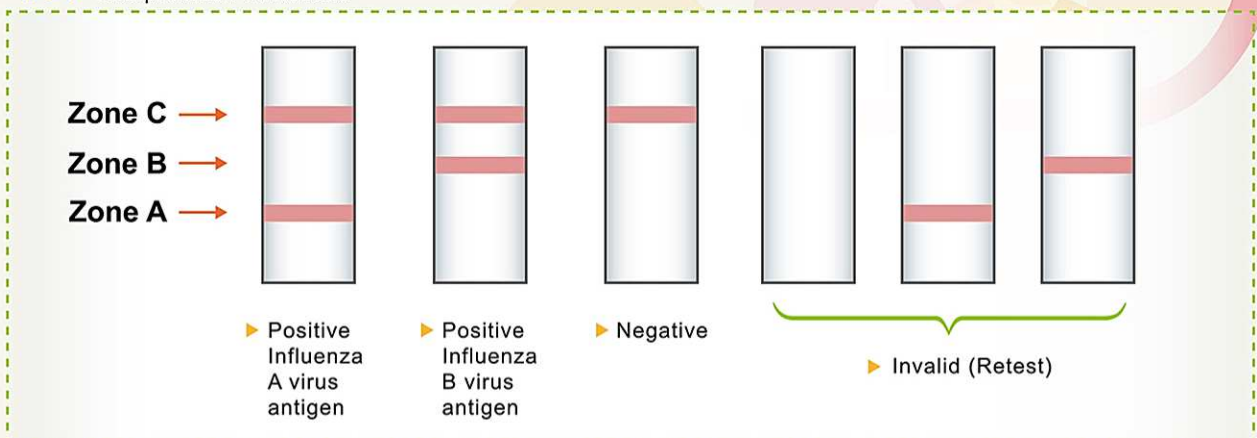

▶ Order Information

Article Number	B0510
Description	20 tests/kit

IN FUNG Co.,Ltd.

• No.61-1, 40th Rd., Taichung Industrial Park, Taichung City 40768, Taiwan R.O.C.
 • +886-4-23550886 • service@infung.com.tw • www.infung.com.tw

