



iF Helicobacter Test
NAME
ID NO
DATE
TIME


iF Helicobacter Test
NAME
ID NO
DATE
TIME
For In Vitro Diagnostic use only




Helicobacter Test

羧芳幽門曲狀桿菌尿素酶測試組


Helicobacter Test

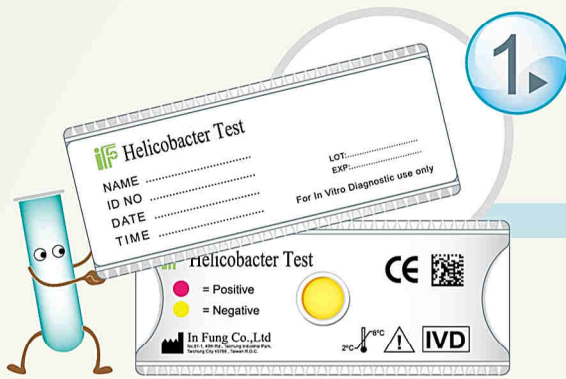


▶ Allow the test slide to reach room temperature and make sure that the gel in the well is yellow before use.

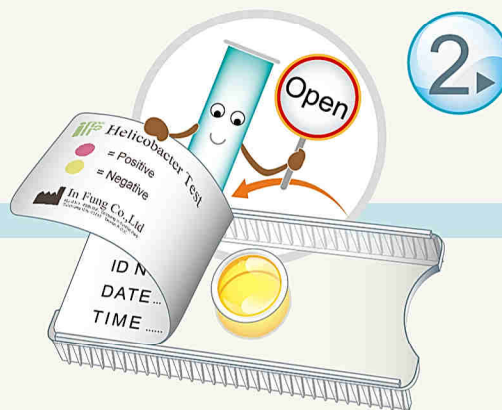


▶ Peel the label back to allow access to the gel without removing the whole label.

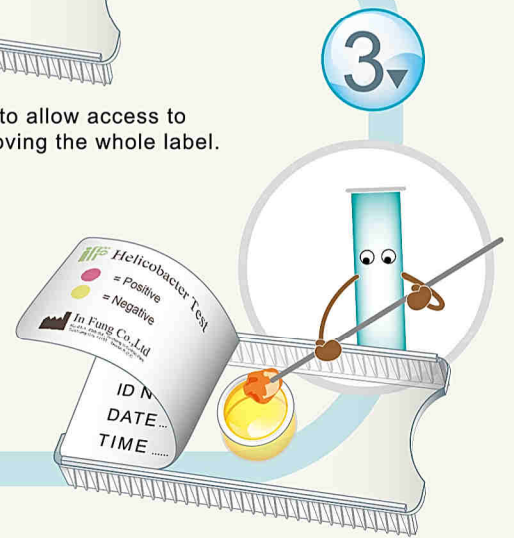


▶ Put the biopsy specimen into the gel.

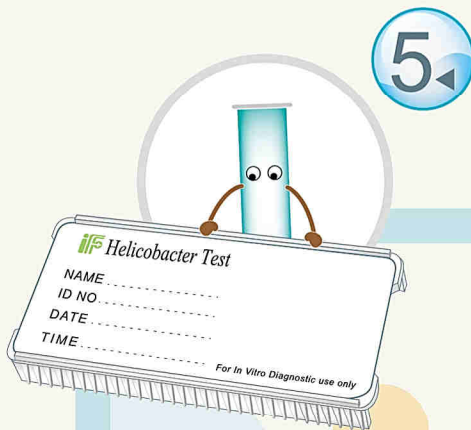

▶ Record patient data.

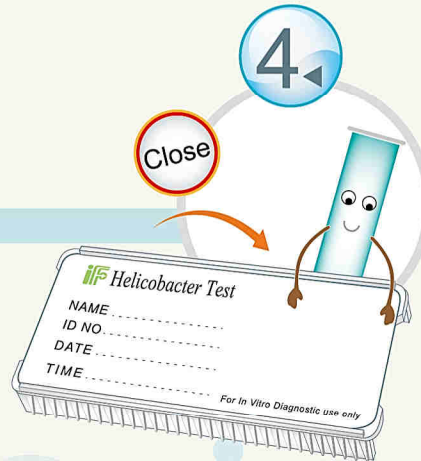

▶ Reseal the test slide to cover the gel.

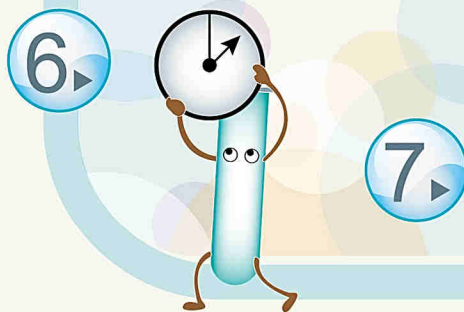

▶ **Rapid Results :**
Incubate the test slide at 37°C for 2 hours.

Normal Results :
Incubate the test slide at room temperature for a maximum of 24 hours.

Interpretation of Results



▶ Order Information

Article Number	C0311
Description	50 tests/kit

▶ Performance Characteristics

Sensitivity	89.01%
Specificity	92.64%

IN FUNG Co.,Ltd.

- No.61-1, 40th Rd., Taichung Industrial Park, Taichung City 40768, Taiwan R.O.C.
- +886-4-23550886
- service@infung.com.tw
- www.infung.com.tw

